COVID-19 Ultra Variant Catcher

Detection of the SARS-CoV-2 S-gene mutations by *Real Time PCR*

REV ST. RT27v2-ENITA.0

REF RT-27v2 - 96 tests

Instructions For Use

INTRODUCTION AND PURPOSE OF USE

COVID-19 Ultra Variant Catcher is a qualitative test that allows the detection, by means of *Real Time PCR*, of the spike protein (*S gene*) mutations L452R, E484K, E484Q and N501Y linked to several Variants of Concern (VOC) and Variants of Interest (VOI) of SARS-CoV-2 virus.

The COVID-19 Ultra Variant Catcher kit must be used with RNA extracted from samples tested as positive for SARS-CoV-2 virus with the current validated, PCR-based methods on the market.

The Procedure allows the detection of the RNA target by means of a reverse-amplification reaction in reaction tubes or microplate wells.

The analysis of the results is performed by means of a Real Time PCR analyzer instrument (thermal cycler integrated with a system for fluorescence detection).

CONTENT

The kit contains sufficient reagents to perform 96 amplification tests.

	Quantity	Description
R1	2 x 290 µl	Amplification mMix dNTPs, Tris-HCl, KCl, MgCl ₂ , Taq Polymerase, <i>AmpErase</i> Uracil N-Glycosylase (<i>UNG</i>) Nuclease-free water, ROX (Pink Cap)
R2	2 x 550 µl	Ultra Variant Catcher probe Mix Upstream primers, downstream primers, Target probes WT (FAM), Target probe mutation L452R (Cy5), Target probe mutation E484K and E484Q (HEX), Target probe mutation N501Y (TexasRed), Nuclease-free water, (White Cap)
R3	2 x 35 µl	Positive Ctrl Synthetic Nucleic Acids: amplification targets for all primers and probes of the assay (Red Cap)
R4	2 x 50 µl	Negative Control (Natural Cap)

MATERIALS AND INSTRUMENTATION REQUIRED BUT NOT SUPPLIED

Disposable latex powder-free gloves or similar material;

Benchtop microcentrifuge (12,000 - 14,000 rpm);

Micropipettes and Sterile tips with aerosol filter;

Vortex;

Plastic materials (microplate and optical adhesive cover, Rotorgene tubes etc...);

Instruments

The COVID-19 Ultra Variant Catcher kit was developed and validated to be used with the following instruments:

Real Time PCR

- Rotor Gene Q MDx from QIAGEN
- CFX96 from Biorad
- 7500 from Life Technologies

Please ensure that the instruments have been installed, calibrated, checked and maintained according to the manufacturers' instructions and recommendations.

SAMPLES AND STORAGE

The **COVID-19 Ultra Variant Catcher** kit must be used with RNA extracted from biological samples that were found positive with the current validated, PCR-based methods on the market for SARS-CoV-2 Virus.

Store the RNA sample at -80°C.

PRECAUTIONS FOR USE

This kit is for in vitro diagnostics (IVD), for professional use only and not for in vivo use.

After reconstitution, the amplification master mix must be used in one time. Repeated thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The reagents should be frozen in aliquots, if they are to be used intermittently. Always follow Good Laboratory Practice (GLP) guidelines.

Wear protective clothing such as laboratory coats and disposable gloves while assaying samples.

Avoid any contact between hands and eyes or nose during specimen collection and testing.

Handle and dispose all used materials into appropriate bio-hazard waste containers that must be discarded according to local laws.

Perform extraction and preparation of reagents in separate settings.

Never pipette solutions by mouth.

Avoid the air bubbles during the master mix dispensing. Eliminate them before starting amplification.

Wash hands carefully after handling samples and reagents.

Do not mix reagents from different lots.

Provided reagents are not infectious and hazardous for the health (see Material Safety data Sheet – MSDS).

Do not eat, drink or smoke in the area where specimens and kit reagents are handled.

Read carefully the instructions for use before using this test.

Do not use beyond the expiration date which appears on the package label.

In case of damaged package, contact the technical support before using the kit.

LIMITS OF THE METHOD

The extreme sensitivity of gene amplification may cause false positives due to cross-contamination between samples and controls. Therefore, you should:

- physically separate all the products and reagents used for amplification reactions from those used for other reactions, as well as from post-amplification products;
- use tips with filters to prevent cross-contamination between samples;
- use disposable gloves and change them frequently;
- carefully open test tubes to prevent aerosol formation;
- close every test tube before opening another one.

The proper functioning of the reverse amplification mix depends on the correct collection, correct transportation, correct storage and correct preparation of a biological sample.

As with any diagnostic device, the results obtained with this product must be interpreted taking into consideration all the clinical data and other laboratory tests available for the patient.

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positive or false negative results.

STORAGE AND STABILITY

Store the product **Ultra Variant Catcher** at -20°C.

The **Ultra Variant Catcher** kit is shipped in dry ice. The kit components should be frozen.

If one or more components are not frozen upon receipt or if the tubes have been damaged during transport, contact Clonit for assistance.

An intact and correctly stored product has a stability of 12 months from the date of production, when there are no limitations due to single components or raw materials. Do not use beyond the expiration date which appears on the package label.

Repeated thawing and freezing of reagents (more than twice) should be avoided, as this might affect the performance of the assay. The reagents should be frozen in aliquots if they are to be used intermittently.

ANALYTICAL PROCEDURE

SOFTWARE SETTING

Lifetechnologies 7500 fast

Turn the instrument and the computer on and open the control software. Click on "Advanced Setup": by default the software will show the "experiment properties" page. Write a file name in the "experiment name" field, choose the type of instrument (7500 or 7500fast), the type of reaction (quantitation standard curve), the type of reagents used (Taqman® Reagents) and the reaction time of analysis (Standard ≈ 2 hours to complete a run).

Open the page named "page setup" ("Define Target and Samples" tab).
In the window "Define Targets" set:

<u>Target</u>	<u>Reporter</u>	<u>Quencher</u>	<u>Note</u>
Wild Type targets	FAM	NONE	non mutated L452, E484 and N501 positions
Mutation L452R	CY5	NONE	
Mutation E484K or E484Q	VIC	NONE	E484 position mutated either in K or in Q
Mutation N501Y	TexasRed	NONE	

Set the samples' names in the window "**Define Samples**".

In the same "**plate setup**" page select the tab "**Assign Target and Samples**". On the screen you will see the microplate map.

Select the area of the plate that will be used for testing and set all the targets.

Select a well dedicated to **Positive Ctrl**. In the space "Assign target to selected wells" choose "Positive control" for all the targets.

Select a well dedicated to the **Negative Control**. In the space "Assign target to selected wells" choose "Negative (N) task" for all the targets.

Select an area of the plate containing the wells dedicated to the **samples** and set all the targets. Associate a sample being analyzed with each well using the "Assign samples to selected wells" window.

For each sample, select the "task **UnKnown (U)**" for the targets in the appropriate space "Assign targets to selected wells".

Set **NONE** as passive reference.

Open the "**Run Method**" page (sheet Graphic View) and set the correct thermal cycle by selecting "**collect data**" in the annealing phase at 60°C

<u>cycles</u>	<u>UNG</u>	<u>Reverse transcription</u>	<u>Denaturation</u>	<u>Annealing</u>
1	25°C 2 min	50°C 15 min		
1			95°C 2 min	
45			95°C 3 sec	60°C 30 sec

In the window "**Reaction volume plate per well**" set a volume of 20 µl.

After preparing the plate and correctly inserting it in the instrument, press the button "Start Run".

Rotor Gene Q MDx

The experiments can be set using the **Quick Start Wizard** or the **Advanced Wizard**, which shows up when the software is started.

Select the wizard "**Advanced**". As a first step, select the model "**Two Step Reaction**" with a double click in the "**New Run**".

In the next window, select the type of rotor installed on the instrument from the list that appears. Check the "**Locking Ring Attached**", check the checkbox and then click "**Next**".

Enter the name of the operator and the reaction volume of 20µl, and then click "Next".

In the next window click on "edit profile". Set the following thermal cycle:

<u>cycles</u>	<u>UNG</u>	<u>Reverse transcription</u>	<u>Denaturation</u>	<u>Annealing</u>
1	25°C 2 min	50°C 15 min		
1			95°C 2 min	
45			95°C 3 sec	60°C 30 sec

Select the annealing from the thermal profile and click on "**Acquiring A to cycling**".

In the next window, select Yellow, Orange and Red from the available channels and add it to **acquiring channel** along with the Green channel and click "**OK**".

Click on "Edit Gain" button and set the following values for each channel:

Target	Reporter	Gain	Note
Wild type L452, E484, N501	Green	5	non mutated L452, E484 and N501 positions
Mutation L452R	Red	5	
Mutation E484K or E484Q	Yellow	5	E484 position mutated either in K or in Q
Mutation N501Y	Orange	5	

In the next window click on "OK" and then click "**Next**".

To start the run, click on the button "**Start Run**". You can save the model before you begin your run by clicking on "**Save Template**".

After clicking on the button "**Start Run**" window appears "**Save As**". The run file can be saved in the desired position by the user.

Once the run started, the window "**Edit Samples**" allows you to set the name of samples and controls in the positions in which they were loaded on the instrument.

Select the position where Positive Control has been placed and name it as **Positive Ctrl**. By clicking on the corresponding "Type" box, in the "Samples" drop-down menu it is possible to select the type of sample being analyzed. Select Positive Control.

Select the position where the Negative Control has been placed and name it as **Negative Control**. By clicking on the corresponding "Type" box, in the "Samples" drop-down menu it is possible to select the type of sample being analyzed. Select "Negative Controls"

Select the position of each individual sample and enter the patient name or code. By clicking on the corresponding "Type" box, in the "Samples" drop-down menu it is possible to select the type of sample being analyzed. Select "**Unknown**"

At the end of the operation click "OK" in the "edit samples" window and wait for the end of the run for the analysis (see "Interpretation of results").

CFX96 Real Time PCR

Turn the instrument and the computer on and start the control software. In the principal screen will appear the window "**Startup wizard**": select "**CFX96**" and press "ok". In the next window push "**create new**" and set the thermal protocol, the reaction volume (20µl) and selecting "**collect data**" in the annealing phase at 60°C

cycles	UNG	Reverse trascription	Denaturation	Annealing
1	25°C 2 min	50°C 15 min		
1			95°C 2 min	
45			95°C 3 sec	60° C 30 sec

Save the protocol and click "**Next**". The software will open in default the sheet "**plate**". Click "**create new**", select "**Fluorophores**" button to choose fluorophores (FAM, HEX, TexasRed and CY5).

Target	Reporter	Note
Wild type L452, E484, N501	FAM	non mutated L452, E484 and N501 positions
Mutation L452R	CY5	
Mutation E484K or E484Q	HEX	non mutated L452, E484 and N501 positions
Mutation N501Y	TexasRed	

Select the well containing **Positive Ctrl** and choose "Sample Type" from the drop-down menu: Positive Control. Click "Load check boxes" to load all the chosen fluorophores and write or select the Target Name.

Select the well containing the **Negative Control** and choose from the drop-down menu "Sample Type": NTC. Click "Load check boxes" to load all the chosen fluorophores and write or select the Target Name.

Select the wells containing the **samples** under test and choose from the drop-down menu "Sample Type": Unknown. Click "Load check boxes" to load all the chosen fluorophores and write or select the Target Name. Save the plate by clicking the "Next" button and, as soon as the plate has been prepared and correctly inserted into the instrument, press the "Start Run" button.

PREPARATION OF THE REACTIONS:

Manual Procedure

Thaw a tube of **Amplification Mmix**;

Thaw a tube of **Ultra Variant Catcher probes Mix**;

Add 275µl of Amplification Mmix directly into the tube of Ultra Variant Catcher probes Mix containing 550 µl. Mix carefully by vortexing (the mixture thus produced is sufficient to prepare 48 reactions: 1 positive control, 1 negative control and 46 samples).

Distribute, in the amplification plate, 15µl of the just reconstituted mix in chosen positions as previously set in the instrument software.

Add 5µl of the solution taken from the **negative control** vial in the negative control position.

Distribute, in chosen positions for each sample, 5µl of the corresponding sample.

Add 5µl of the solution taken from the **Positive Ctrl** vial in the positive control position.

Carefully seal the plate using optical adhesive films and check that, in the mix, there are no air bubbles that could interfere with the amplification.

For the Rotorgene Q 5/6 plex instrument, carefully seal each tube with the appropriate caps. The presence of air bubbles is not important: the centrifugal force of the rotor will allow their automatic elimination.

Transfer the plate to the instrument and press the "Run" button.

QIASymphony AS Plate Setup.

Samples processed on the QIASymphony SP can be transferred automatically to the QIASymphony AS (integrated operation) to the Rotor-Gene Q tubes or 96 wells MicroPlate

Thaw a tube of **Amplification Mmix**;

Thaw a tube of **Ultra Variant Catcher probes Mix**;

Add 275µl of Amplification Mmix directly into the tube of Ultra Variant Catcher probes Mix probes Mix containing 550 µl. Mix carefully by vortexing (the mixture thus produced is sufficient to prepare 48 reactions: 1 positive control, 1 negative control and 46 samples).

Insert the obtained Master Mix tubes in the correct position inside the QIASymphony AS;

Thaw the **Positive Ctrl** and insert it in the required position of **QIASymphony AS** Thaw a tube of **Negative Control** and insert it in the required position of **QIASymphony AS**;

Follow the instructions of *QIASymphony AS*.

Select the Assay Parameter Set to use in the run

Assay Parameter Set
APS_15+5 (user-prep_MM)

Once the preparation is finished, press "Remove" in the assay setup "Overview" screen. Open the "Assays" drawer and unload the assayrack(s). Download the result and cycler files and proceed to protocol on the Real Time Instrument (RotorGene Q MDx).

QUALITATIVE ANALYSIS

Lifetechnologies 7500 Fast

At the end of the PCR run, the software automatically opens the "Analysis" window in the "**Amplification plot**" sheet on the menu on the left.

Select the wells corresponding to the positive control, negative control and samples for analysis.

Select in the "**Option**" window inside the "**Target**" pop-up menu the **WT** (for wild type L452, E484 and N501) target.

Check the correct setting of the threshold.

Repeat the steps above for each of the targets **Mutation L450R**, **Mutation E484K** or **E484Q**, and **Mutation N501**.

The analysis of the results is made selecting from the menu in the left the page "**Analysis**". From the page "**Amplification Plot**", maintaining open the sheet "view well plate" in the right side of the software select the wells containing the **Positive Ctrl** and the **Negative Control** and verify the parameters described in the paragraph "Interpretation of the Results".

At the page "**Amplification Plot**" check the amplification plot for every single sample.

By opening the sheet "**view well table**" in the right side of the software it is possible to verify the Threshold Cycle and emitted fluorescence values for each well.

By selecting the option "**export**" from the main menu, the window "**export properties**" will open. Indicate the file name, select the position to save it (**Browse**) and click on button "**Start export**". In this way the software will permit to save a excel file with all the data corresponding to selected experiment.

Rotor Gene Q MDx

At the end of the PCR run open the "Analysis" window. Select the "**Quantification**" sheet and click on "**cycling A (green)**".

Select from the menu "**Dynamic Tube**" and subsequently "**Slope correct**". If necessary, reduce the background noise of non-specific signals by selecting the "**Outlier Removal**" option and entering a value between 5% and 30%.

Check the correct setting of the threshold in the space provided "**CT calculation – Threshold**".

Repeat the steps above for each of the fluorescence channels "**cycling A (yellow)**", "**cycling A (orange)**", and "**cycling A (red)**".

It is possible to do a second type of analysis that shows simultaneously all the fluorescence together: in the "Analysis" window select "Other" and then "Allelic Discrimination"; Select together all the channels available and click "Show".

Select from the menu "Dynamic Tube" and subsequently "Slope correct": the different channels will be shown with different types of markers on the curves (No markers, Circles, Squares, and Crosses). If necessary, reduce the background noise of non-specific signals by selecting the "Outlier Removal" option and entering a value between 5% and 30%.

You can print a report of the analysis by clicking on the "Report" window and selecting the file in the first Quantification cycling A (green) and then the file cycling A (yellow), cycling A (orange) and cycling A (red).

CFX96 Real Time PCR System

At the end of the PCR, select the "quantitation" sheet. On the top of the screen, select "settings" from the menu and choose "Baseline Threshold" for each fluorophore.

For more accurate results, it is essential to set separately the threshold for each fluorophore of the amplification mix.

You can export the report by selecting notepad icon on the top of the screen.

INTERPRETATION OF RESULTS

In the amplification reaction of each sample, the Ct values of each specific probe for L452R, E484K, E484Q and N501Y mutations and respective Wild Type sequence are used to detect the potential presence of the SARS-Cov-2 variants containing those mutations.

An increase of the fluorescence of the specific probes for WT sequences (FAM) indicates the absence of mutations in the investigated S-gene positions. On the contrary, an increase of the fluorescence of the specific probes for the L452R mutation (Cy5), E484K or E484Q mutation (HEX) and N501Y mutation (TexasRed) indicates the potential presence of a VOC or VOI that contains those mutations, as indicated in the following tables:

VARIANTS OF CONCERN (VOC)						
WHO label	Pango lineage	Spike mutation of Interest	Detected fluorescence signals			
			FAM	Cy5	HEX	TexasRed
Alpha	B.1.1.7	N501Y	X			X
	B.1.1.7 + E484K	N501Y, E484K	X		X	X
Beta	B.1.351	N501Y, E484K	X		X	X
Gamma	P.1	N501Y, E484K	X		X	X
Delta	B.1.617.2	L452R	X	X		

VARIANTS OF INTEREST (VOI)						
WHO label	Pango lineage	Spike mutation of Interest	Detected fluorescence signals			
			FAM	Cy5	HEX	TexasRed
Epsilon	B.1.427/B.1.429	L452R	X	X		
Zeta	P.2	E484K	X		X	
Eta	B.1.525	E484K	X		X	
Theta	P.3	E484K, N501Y	X		X	X
Iota	B.1.526	E484K	X		X	
Kappa	B.1.617.1	L452R, E484Q		X	X	

The assay cannot differentiate between E484K and E484Q mutations. A signal in the VIC/HEX/Yellow channel indicates the presence of any of the two mutations.

As with any diagnostic device, the results obtained with this product must be interpreted taking in consideration all the clinical data and the other laboratory tests available for the patient.

As with any diagnostic device, with this product there is a residual risk of obtaining invalid, false positives or false negatives results.

The use of positive and negative controls in each amplification session allows to verify the correct functioning of the amplification mix and the absence of any contamination.

The instrument software is able to analyze the emitted fluorescence of specific probes for WT sequences (FAM), L452R mutation (Cy5), E484K or E484Q mutation (HEX) and N501Y mutation (Texas Red).

A correct functioning of the amplification mixture must be verified by analyzing the following parameters:

Parameter	Reference
Positive Ctrl	Ct ≤ 30 for FAM, HEX, Texas Red and Cy5

If the results of the amplification reaction of the positive controls produce a Ct > 30 or undetermined for any of the fluorescence signals, the run cannot be considered valid and must be repeated.

PERFORMANCE EVALUATION

Analytical sensitivity:

The COVID-19 Ultra Variant Catcher kit must be used with RNA extracted from positive samples for SARS-CoV-2 virus that have shown amplification within the 30th amplification cycle in a previous test with the current PCR-based methods validated on the market.

Parameter	Reference
SARS-CoV-2 Positive RNA	Ct ≤ 30

Clinical sensitivity:

The Clinical Sensitivity is defined as the ability of the method to detect true positive samples. The clinical sensitivity was evaluated by analyzing samples positive to the variants of SARS-CoV-2 virus and the test was performed following the method recommendations. Results were confirmed with another method available on the market.

Obtained results show a clinical sensitivity of 100% for extracted RNA samples.

Diagnostic Specificity:

The Diagnostic Specificity is defined as the ability of the method to detect true negative samples. The diagnostic specificity was evaluated by analyzing samples negative to the variants of SARS-CoV-2 virus and confirmed as negative with another method available on the market.

Diagnostic specificity is 100% for extracted RNA samples.

Analytical Specificity:

Analytical specificity is guaranteed by the use of specific primers and probes designed for SARS-CoV-2 L452R, E484K, E484Q and N501Y mutations and the respective Wild Type sequences.

The alignment of the selected regions for specific primers hybridization for SARS-CoV-2 with available sequences of the region X in database, demonstrated: their conservation, the absence of significant mutations for the analyzed target.

Interferences:

Verify that in the RNA extracted from the sample there is no contamination from mucoproteins and hemoglobin, to exclude possible inhibition of PCR reaction. The interference due to contaminants can be detected through a spectrophotometric analysis, verifying the ratio between the absorbance readings at 260 nm (maximum absorption of Nucleic Acids) and 280 nm (maximum absorption of Proteins). A pure RNA should have a ratio of approximately 2.

QUALITY CONTROL

It is recommended to include in each analytical run, as quality control of every extraction, amplification and detection step, an already tested negative and positive sample, or a reference material with known concentration.

In accordance with the Clonit srl ISO EN 13485 Certified Quality Management System, each lot of COVID-19 Ultra Variant Catcher is tested against predetermined specifications to ensure consistent product quality.

BIBLIOGRAPHY

European Centre for Disease Prevention and Control. Rapid increase of a SARS-CoV-2 variant with multiple spike protein mutations observed in the United Kingdom – 20 December 2020. ECDC: Stockholm; 2020.

Emergencies preparedness, response SARS-CoV-2 Variant – United Kingdom of Great Britain and Northern Ireland Disease Outbreak News 21 December 2020

SARS-CoV-2 variant of concern as of 24 May 2021. European Center for Disease Prevention and Control.

Tracking SARS-CoV-2 variants. 08/06/2021 World Health Organization

grinch | global report investigating novel coronavirus haplotypes: B.1.1.7 report 2021-02-08

grinch | global report investigating novel coronavirus haplotypes: B.1.351 report 2021-02-08

grinch | global report investigating novel coronavirus haplotypes: P.1 report 2021-02-08

grinch | global report investigating novel coronavirus haplotypes: B.1.617.2 report 2021-05-19

TECHNICAL ASSISTANCE

For any question and support, please contact our Technical support:

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CLONIT S.r.l.